
ORBIS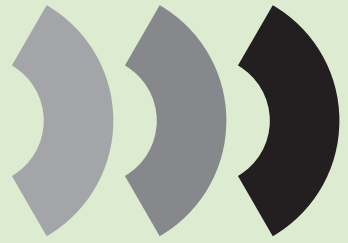

Alginate Plus

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For Dental Use Only

Dosing and mixing

- Open the bag and pour Alginat Plus into an airtight container.
- Keep the lot number and expiry date of the product by cutting the relevant details off the bag. Shake two or three times to disperse the powder then proceed with the mixing. Using the dosing spoon, remove the powder without compressing it, levelling off or removing any excess of Alginat Plus by passing a flat spatula over the top of the spoon.
- Pour the powder into a rubber cup. Repeat the operation until the required quantity is reached.
- Add 1/3 of a measure of water (18 ml) for every spoon full of powder (25 ml). E.g. for 2 measures of powder (50 ml), equal to one impression tray, add 2/3 of a measure of water (36 ml).
- Close the container and store it at 5-27 °C/41-80 °F.
- Add the water to the powder and mix for the times specified in the technical data here below.
- Spread the material over the impression tray and insert into the mouth before the end of the working time.
- Wait for the material to harden until the end of the recommended time in mouth.

Cleaning and disinfection of the impression

After taking the impression, carefully rinse it in order to remove any traces of saliva, drain off the excess of water and disinfect immediately, always follow the manufacturer's instructions.

Pouring the impression – Recommended dental stones

We recommend that you pour the impression immediately using dental stones type 3 or type 4.

Storing the impression

If the gypsum cannot be immediately poured, drain off the excess of water and store the impression in a hermetically sealed bag at room temperature (23 °C/73 °F). Under these conditions the gypsum pouring can be deferred for up to 48 hr after the impression taking.

Times for clinical use

Mixing Time	30''
Working Time	1'10''
Time in mouth (minimum)	1'00''
Setting Time	2'10''
Total Working Time (ISO 1563)	2'00''

© 2011-08-01 (Version -03)

* Times for clinical use are indexed to the start of mixing with deionised water at 23°C/73°F. Colder water retards setting, warmer water speeds up setting. Variations in water hardness may result in deviations in times. According to ADA 18-ISO 1563 specifications.

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